




Rodwell AUTOCLAVE Products


For use in monitoring steam sterilisation processes.

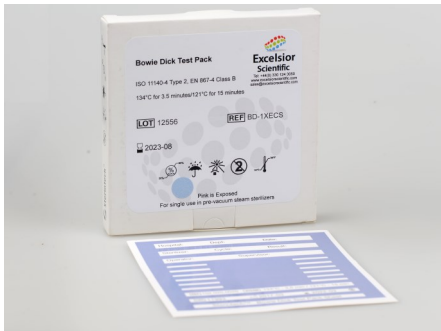

Bowie & Dick Tests

A Bowie & Dick test is a test for steam penetration in dynamic air removal steam sterilisers. A daily steam penetration test should be run in every autoclave. The colour of the indicator test sheet changes from Blue to Pink which will confirm that the steam penetration has been effective up to the centre of the sterilisation pack.

B&D test Steam: 134 C, 3,5min.
Conforms to EN-867-4, ISO 11140-1, class 2
Catalogue No. AC980 (Case of 20)

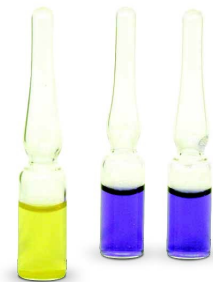

Chemical Indicators

Chemical Process Indicators are self-adhesive labels suitable for application on various types of autoclave load. The label colour indicates at a glance whether or not the product has gone through the sterilisation process, making it easy for sterilisation facilities to ensure that products have been exposed to the sterilisation process. Labels turn from blue to pink after exposure to 121 C for 3-10 mins, or 134 C for 0.5-2 mins.

Catalogue No. AC986



Biological Indicators



Fast and no fuss. Rodwell offers regular biological indicators as well as self contained biological indicators. Simply place the SCBI into the autoclave, remove after cycle end, crush the cap (or squeeze the side for Mini SCBIs) and incubate

for 24 hours. A negative result ensures the micro-organism within has been destroyed and guarantees sterility of the process.

Catalogue No. AC982 (Regular), AC984 (Mini)

Catalogue No. AC983 (Glass Ampoules)




Autoclave TST Strips:

The TST (Time, Steam and Temperature) Steriliser Control is the most sophisticated Chemical device produced, being an integrated control system that registers a Pass only when it has been subjected to the required heat and time ratio in saturated heat steam. when exposed to steam for 15 minutes at 121 C.

Can be placed throughout the load for thorough checking.

Catalogue No. AC981

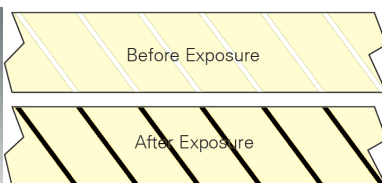

Autoclave Tape

White adhesive tape, for use in standard autoclaves. Transitions from white to dark after 15 mins at 121 C. Tape Size: 18mm x 55m,

Catalogue No. AC985

For Further information please contact The Rodwell Autoclave Company